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Test Name: NN **BiLD** Test Code: IL_ **BiLD-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ **МКМОЛЬ/л** * **Decimal Points** 1
☐ User Define
Conv.Factor: 0.000

Methodology
Type
☒ End Point ☐ Rate
Measuring Point
16 - ☒ 33
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary 546 **Secondary** 700
Correction Constant
Slope 1.000 **Intercept** 0.000

Test Name: NN **BiLD** Test Code: IL_ **BiLD-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	16.0	0.0	0.0
2	8.0	0.0	0.0
3	30.0	0.0	0.0
4	0.0	0.0	0.0

Diluent **Diluent Warning Limit** 0 Tests

Reagent Volume
R1 **BiLD-d1** **R2** **BiLD-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	160	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	40	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

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Ranges		Limits																																										
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>0.0</td><td>3.4</td></tr><tr><td>Female</td><td>0.0</td><td>3.4</td></tr><tr><td>Other</td><td>0.0</td><td>3.4</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	0.0	3.4	Female	0.0	3.4	Other	0.0	3.4	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0		
Normal Range																																												
	Lower	Upper																																										
Male	0.0	3.4																																										
Female	0.0	3.4																																										
Other	0.0	3.4																																										
Reaction Slope	Absorbance Limit																																											
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																											
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																									
Non-Linear Limit																																												
0	%																																											
Prozone Limit																																												
<input type="radio"/> Above <input checked="" type="radio"/> Below																																												
<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none																																								
Limit	Equation																																											
0.0	none																																											
<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																																									
Judge Point																																												
0																																												

	User Range			------------	-------		Lower	Upper		0	170				
	ValidRange			------------	-------		Lower	Upper		0	170				
Qualitative															
☐ On ☒ Off															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																			
NN	BiLD	IL_	BiLD-d																										
Calibration				Condition				Calibration/R-Blank Limit																					
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>1-point</td><td></td></tr></thead><tbody></tbody></table>				Method		1-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>Linear</td><td></td></tr></thead><tbody></tbody></table>				Curve Type		Linear		<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				R-Blank Limit		2500.0	mAbs						
Method																													
1-point																													
Curve Type																													
Linear																													
R-Blank Limit																													
2500.0	mAbs																												
<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>				Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>				Stability			days	<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>20.0</td><td>%</td></tr></thead><tbody></tbody></table>				Cal Reps Range		20.0	%						
Repeats																													
2																													
Stability																													
	days																												
Cal Reps Range																													
20.0	%																												
<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td></td><td></td></tr></thead><tbody></tbody></table>				No				<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>50.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>				Min Cal Reps		50.0	mAbs	<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>20</td><td>%</td></tr></thead><tbody></tbody></table>				Factor Change		20	%						
No																													
Min Cal Reps																													
50.0	mAbs																												
Factor Change																													
20	%																												
<table border="1"><thead><tr><th colspan="2">Calibrator</th></tr><tr><th></th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>TruCalU **</td></tr><tr><td>2</td><td></td></tr><tr><td>3</td><td></td></tr><tr><td>4</td><td></td></tr><tr><td>5</td><td></td></tr></tbody></table>				Calibrator			Conc	1	TruCalU **	2		3		4		5		<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>				M-Point Curve Fit		0.0	%				
Calibrator																													
	Conc																												
1	TruCalU **																												
2																													
3																													
4																													
5																													
M-Point Curve Fit																													
0.0	%																												
<table border="1"><thead><tr><th colspan="2">Condition</th></tr><tr><td><input checked="" type="checkbox"/> Reagent Blank</td><td></td></tr><tr><td><input type="checkbox"/> Auto Reagent Blank by Bottle</td><td></td></tr><tr><td><input type="checkbox"/> Ask for calibration when reagent lot changes</td><td></td></tr></thead><tbody></tbody></table>				Condition		<input checked="" type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes																			
Condition																													
<input checked="" type="checkbox"/> Reagent Blank																													
<input type="checkbox"/> Auto Reagent Blank by Bottle																													
<input type="checkbox"/> Ask for calibration when reagent lot changes																													

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