

Цистатин - C

ИтипоTD (Иммунотурбодиметрический с латексными частицами)

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **CysC** Test Code: IL_ **CysC-d** **Measure** Sample Reagent Ranges Limits Calibration

Sample Type
 Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ Мг/л* Decimal Points: 2
☐ User Define
 Conv.Factor: 0.000

Methodology
 Type: ☐ End Point ☒ Rate
 Measuring Point: 19 - ☒ 31
 Photometric: ☒ 1 Wavelength ☐ 2 Wavelength
 Primary: 510 Secondary:

Correction Constant
 Slope: 1.000 Intercept: 0.000

Test Name: NN **CysC** Test Code: IL_ **CysC-d** **Measure** **Sample Reagent** Ranges Limits Calibration

Sample Volume **Sampling Condition**

	Sample	Dilution		Diluent	Condition No.	1	2	3	4
	Volume	Sample Vol.	Diluent Vol.						
1	2.0	0.0	0.0		First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
2	2.0	30.0	60.0		Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
3	4.0	0.0	0.0		Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
4	0.0	0.0	0.0		Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Panic H	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					<input checked="" type="checkbox"/> User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
					<input checked="" type="checkbox"/> HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
					<input checked="" type="checkbox"/> ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Diluent
 Diluent Warning Limit: 0 Tests

Reagent Volume
 R1: CysC-d1 R2: CysC-d2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	180	0.0	<input checked="" type="checkbox"/>	10	No Control
R2	60	0.0	<input checked="" type="checkbox"/>	10	No Control

Цистатин - С

ИттипоTD (Иммунотурбодиметрический с латексными частицами)

Ranges			Limits																																																																																							
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.53</td> <td>1.02</td> </tr> <tr> <td>Female</td> <td>0.53</td> <td>1.02</td> </tr> <tr> <td>Other</td> <td>0.53</td> <td>1.02</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0.53	1.02	Female	0.53	1.02	Other	0.53	1.02	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>100</td> <td></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>2600.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table> </td> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>			Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>100</td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		Value	%	100		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>2600.0</td> <td>none</td> </tr> </tbody> </table>		Prozone Limit		Value	Equation	2600.0	none	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value		0		<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table>			Qualitative		On	Off	<input type="radio"/>	<input checked="" type="radio"/>
Normal Range																																																																																										
	Lower	Upper																																																																																								
Male	0.53	1.02																																																																																								
Female	0.53	1.02																																																																																								
Other	0.53	1.02																																																																																								
Reaction Slope		Absorbance Limit																																																																																								
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>100</td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		Value	%	100		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>2600.0</td> <td>none</td> </tr> </tbody> </table>		Prozone Limit		Value	Equation	2600.0	none																																																																											
Non-Linear Limit																																																																																										
Value	%																																																																																									
100																																																																																										
Prozone Limit																																																																																										
Value	Equation																																																																																									
2600.0	none																																																																																									
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th></th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value		0																																																																
User Range																																																																																										
Lower	Upper																																																																																									
0.01	8.0																																																																																									
Serum Index Limits																																																																																										
	Limit																																																																																									
Hemolysis	0.0																																																																																									
Icterus	0.0																																																																																									
Lipemia	0.0																																																																																									
Judge Point																																																																																										
Value																																																																																										
0																																																																																										
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table>			Qualitative		On	Off	<input type="radio"/>	<input checked="" type="radio"/>																																																																									
ValidRange																																																																																										
Lower	Upper																																																																																									
0.01	8.0																																																																																									
Qualitative																																																																																										
On	Off																																																																																									
<input type="radio"/>	<input checked="" type="radio"/>																																																																																									

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																							
NN	CysC	IL_	CysC-d																																														
Calibration				Condition				Calibration/R-Blank Limit																																									
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>5P Log</td> <td></td> </tr> </tbody> </table>				Method		Curve Type		M-point		5P Log		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>				Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td>6</td> <td></td> </tr> </tbody> </table>				No		6																			
Method		Curve Type																																															
M-point		5P Log																																															
Repeats		Stability																																															
2			days																																														
No																																																	
6																																																	
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl-0.9</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-CysC-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-CysC-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-CysC-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-CysC-4</td> <td>**</td> </tr> <tr> <td>6</td> <td>TC-CysC-5</td> <td>**</td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	NaCl-0.9	0.0	2	TC-CysC-1	**	3	TC-CysC-2	**	4	TC-CysC-3	**	5	TC-CysC-4	**	6	TC-CysC-5	**	<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		<input type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <th>Value</th> <th>mAbs</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table>				R-Blank Limit		Value	mAbs	2500.0	
Calibrator																																																	
	Calibrator	Conc																																															
1	NaCl-0.9	0.0																																															
2	TC-CysC-1	**																																															
3	TC-CysC-2	**																																															
4	TC-CysC-3	**																																															
5	TC-CysC-4	**																																															
6	TC-CysC-5	**																																															
Condition																																																	
<input type="checkbox"/>	Reagent Blank																																																
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																																
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																																
R-Blank Limit																																																	
Value	mAbs																																																
2500.0																																																	
				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>15.0</td> <td></td> </tr> </tbody> </table>				Cal Reps Range		Value	%	15.0		<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <th>Value</th> <th>mAbs</th> </tr> </thead> <tbody> <tr> <td>50.0</td> <td></td> </tr> </tbody> </table>				Min Cal Reps		Value	mAbs	50.0																											
Cal Reps Range																																																	
Value	%																																																
15.0																																																	
Min Cal Reps																																																	
Value	mAbs																																																
50.0																																																	
				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>35</td> <td></td> </tr> </tbody> </table>				Factor Change		Value	%	35		<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td></td> </tr> </tbody> </table>				M-Point Curve Fit		Value	%	0.0																											
Factor Change																																																	
Value	%																																																
35																																																	
M-Point Curve Fit																																																	
Value	%																																																
0.0																																																	

Цистатин - С

ИттипоTD (Иммунотурбодиметрический с латексными частицами)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0 + 5 ур.) Spline или 5P Log, калибратор TruCal Cystatin C.

В качестве нулевой точки используйте физраствор.

Контроль TruLab Cystatin C уровень 1 и 2.