

Ферритин (Ferritin)

ImmunoTD (Иммунотурбидиметрический, усиленный латексом)

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **Ferritin** Test Code: IL_ **FRN-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type: **Serum**

Reaction Cycle: **Standard** **Extended**

Reporting Unit: **мкг/л*** Decimal Points: **1**

User Define: Conv. Factor: **0.000**

Methodology: Type: **End Point** **Rate**

Measuring Point: **19** - **28**

Photometric: **1 Wavelength** **2 Wavelength**

Primary: **600** Secondary:

Correction Constant: Slope: **1.000** Intercept: **0.000**

Test Name: NN **Ferritin** Test Code: IL_ **FRN-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume:

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	6.0	0.0	0.0
2	2.0	0.0	0.0
3	12.0	0.0	0.0
4	0.0	0.0	0.0

Diluent: Diluent Warning Limit: **0** Tests

Reagent Volume: R1: **FRN-d1** R2: **FRN-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	120	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	60	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition:

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Below N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Above N-Range	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Panic L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Panic H	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	User Range L	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	User Range H	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Noise	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	Prozone	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	ABS!	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Ферритин (Ferritin)

ImmunoTD (Иммунотурбидиметрический, усиленный латексом)

Ranges		Limits																																												
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>30</td><td>400</td></tr><tr><td>Female</td><td>15</td><td>150</td></tr><tr><td>Other</td><td>15</td><td>400</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	30	400	Female	15	150	Other	15	400	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table></td><td><table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead></table></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table></td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table>	Non-Linear Limit		100	%	<table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead></table>	2600.0	mAbs	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table></td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table>		Judge Point	0
Normal Range																																														
	Lower	Upper																																												
Male	30	400																																												
Female	15	150																																												
Other	15	400																																												
Reaction Slope	Absorbance Limit																																													
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																													
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table>	Non-Linear Limit		100	%	<table border="1"><thead><tr><td>2600.0</td><td>mAbs</td></tr></thead></table>	2600.0	mAbs																																							
Non-Linear Limit																																														
100	%																																													
2600.0	mAbs																																													
<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td colspan="2"><table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table></td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table>		Judge Point	0																															
Prozone Limit																																														
<input type="radio"/> Above <input checked="" type="radio"/> Below																																														
<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none																																										
Limit	Equation																																													
0.0	none																																													
<table border="1"><thead><tr><th>Judge Point</th></tr><tr><td>0</td></tr></thead></table>		Judge Point	0																																											
Judge Point																																														
0																																														
<table border="1"><thead><tr><th colspan="2">User Range</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>2</td><td>1200</td></tr></tbody></table>		User Range		Lower	Upper	2	1200	<table border="1"><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																											
User Range																																														
Lower	Upper																																													
2	1200																																													
Serum Index Limits																																														
	Limit																																													
Hemolysis	0.0																																													
Icterus	0.0																																													
Lipemia	0.0																																													
<table border="1"><thead><tr><th colspan="2">ValidRange</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>2</td><td>1200</td></tr></tbody></table>		ValidRange		Lower	Upper	2	1200																																							
ValidRange																																														
Lower	Upper																																													
2	1200																																													
Qualitative																																														
<input type="radio"/> On <input checked="" type="radio"/> Off																																														

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																																							
NN	Ferritin	IL	FRN-d																																																														
Calibration				Condition				Calibration/R-Blank Limit																																																									
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>M-point</td><td></td></tr></thead><tbody><tr><td>Repeats</td><td>2</td></tr><tr><td>No</td><td>5</td></tr></tbody></table>				Method		M-point		Repeats	2	No	5	<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>4P Log</td><td></td></tr></thead><tbody><tr><td>Stability</td><td>days</td></tr></tbody></table>				Curve Type		4P Log		Stability	days	<table border="1"><tbody><tr><td><input type="checkbox"/> Reagent Blank</td><td></td></tr><tr><td><input type="checkbox"/> Auto Reagent Blank by Bottle</td><td></td></tr><tr><td><input type="checkbox"/> Ask for calibration when reagent lot changes</td><td></td></tr></tbody></table>				<input type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes		<table border="1"><tbody><tr><td><table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>50.0</td><td>%</td></tr></thead></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>100.0</td><td>mAbs</td></tr></thead></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>35</td><td>%</td></tr></thead></table></td><td></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead></table></td><td></td></tr></tbody></table>				<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead></table>	R-Blank Limit		2500.0	mAbs		<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>50.0</td><td>%</td></tr></thead></table>	Cal Reps Range		50.0	%		<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>100.0</td><td>mAbs</td></tr></thead></table>	Min Cal Reps		100.0	mAbs		<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>35</td><td>%</td></tr></thead></table>	Factor Change		35	%		<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead></table>	M-Point Curve Fit		0.0	%	
Method																																																																	
M-point																																																																	
Repeats	2																																																																
No	5																																																																
Curve Type																																																																	
4P Log																																																																	
Stability	days																																																																
<input type="checkbox"/> Reagent Blank																																																																	
<input type="checkbox"/> Auto Reagent Blank by Bottle																																																																	
<input type="checkbox"/> Ask for calibration when reagent lot changes																																																																	
<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead></table>	R-Blank Limit		2500.0	mAbs																																																													
R-Blank Limit																																																																	
2500.0	mAbs																																																																
<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>50.0</td><td>%</td></tr></thead></table>	Cal Reps Range		50.0	%																																																													
Cal Reps Range																																																																	
50.0	%																																																																
<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>100.0</td><td>mAbs</td></tr></thead></table>	Min Cal Reps		100.0	mAbs																																																													
Min Cal Reps																																																																	
100.0	mAbs																																																																
<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>35</td><td>%</td></tr></thead></table>	Factor Change		35	%																																																													
Factor Change																																																																	
35	%																																																																
<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead></table>	M-Point Curve Fit		0.0	%																																																													
M-Point Curve Fit																																																																	
0.0	%																																																																
<table border="1"><thead><tr><th colspan="3">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>NaCl*</td><td>0.0</td></tr><tr><td>2</td><td>TC-Ferrit-1</td><td>**</td></tr><tr><td>3</td><td>TC-Ferrit-2</td><td>**</td></tr><tr><td>4</td><td>TC-Ferrit-3</td><td>**</td></tr><tr><td>5</td><td>TC-Ferrit-4</td><td>**</td></tr></tbody></table>				Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-Ferrit-1	**	3	TC-Ferrit-2	**	4	TC-Ferrit-3	**	5	TC-Ferrit-4	**																																									
Calibrator																																																																	
	Calibrator	Conc																																																															
1	NaCl*	0.0																																																															
2	TC-Ferrit-1	**																																																															
3	TC-Ferrit-2	**																																																															
4	TC-Ferrit-3	**																																																															
5	TC-Ferrit-4	**																																																															

Ферритин (Ferritin)

ImmunoTD (Иммунотурбидиметрический, усиленный латексом)



* - Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (5 точек, 0+4 уровня) Spline или 4P Log,
(калибратор TruCal Ferritin).

Контроль TruLab Protein уровень 1 и 2.