

Параметры для ввода в программу анализатора iLab-Taurus

Test Name
NN **Lactat**

Test Code
IL_ **Lact-d**

Measure

Sample Reagent

Ranges Limits

Calibration

Sample Type
Serum

Reaction Cycle
☒ Standard
☐ Extended

Reporting Unit
☒ ммоль/л *

Decimal Points
2

☐ User Define

Conv.Factor
0.000

Methodology

Type
☒ End Point
☐ Rate

Measuring Point
16 - ☒ 33

Photometric
☐ 1 Wavelength
☒ 2 Wavelength

Primary
340
Secondary
546

Correction Constant

Slope
1.000

Intercept
0.000

Test Name
NN **Lactat**

Test Code
IL_ **Lact-d**

Measure

Sample Reagent

Ranges Limits

Calibration

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.5	0.0	0.0
2	2.0	0.0	0.0
3	4.0	0.0	0.0
4	0.0	0.0	0.0

Diluent
Diluent Warning Limit
0 Tests

Reagent Volume

R1
Lact-1
R2
Lact-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	160	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	40	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Ranges		Limits																																																																																						
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.5</td> <td>2.2</td> </tr> <tr> <td>Female</td> <td>0.5</td> <td>2.2</td> </tr> <tr> <td>Other</td> <td>0.5</td> <td>2.2</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	0.5	2.2	Female	0.5	2.2	Other	0.5	2.2	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>13.3</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th>Index</th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>13.3</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Equation</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>	Non-Linear Limit		Value	%	0		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table>	Prozone Limit		Value	%	2500.0		<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>13.3</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.1	13.3	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th>Index</th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits		Index	Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>13.3</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.1	13.3	<table border="1"> <thead> <tr> <th colspan="2">Equation</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Equation		Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table>		Qualitative		On	Off	<input type="radio"/>	<input checked="" type="radio"/>	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value	%	0	
Normal Range																																																																																								
	Lower	Upper																																																																																						
Male	0.5	2.2																																																																																						
Female	0.5	2.2																																																																																						
Other	0.5	2.2																																																																																						
Reaction Slope	Absorbance Limit																																																																																							
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>	Non-Linear Limit		Value	%	0		<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td></td> </tr> </tbody> </table>	Prozone Limit		Value	%	2500.0																																																																												
Non-Linear Limit																																																																																								
Value	%																																																																																							
0																																																																																								
Prozone Limit																																																																																								
Value	%																																																																																							
2500.0																																																																																								
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>13.3</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.1	13.3	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th>Index</th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits		Index	Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																					
User Range																																																																																								
Lower	Upper																																																																																							
0.1	13.3																																																																																							
Serum Index Limits																																																																																								
Index	Limit																																																																																							
Hemolysis	0.0																																																																																							
Icterus	0.0																																																																																							
Lipemia	0.0																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.1</td> <td>13.3</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.1	13.3	<table border="1"> <thead> <tr> <th colspan="2">Equation</th> </tr> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>		Equation		Limit	Equation	0.0	none																																																																									
ValidRange																																																																																								
Lower	Upper																																																																																							
0.1	13.3																																																																																							
Equation																																																																																								
Limit	Equation																																																																																							
0.0	none																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <th>On</th> <th>Off</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/></td> <td><input checked="" type="radio"/></td> </tr> </tbody> </table>		Qualitative		On	Off	<input type="radio"/>	<input checked="" type="radio"/>	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <th>Value</th> <th>%</th> </tr> </thead> <tbody> <tr> <td>0</td> <td></td> </tr> </tbody> </table>		Judge Point		Value	%	0																																																																										
Qualitative																																																																																								
On	Off																																																																																							
<input type="radio"/>	<input checked="" type="radio"/>																																																																																							
Judge Point																																																																																								
Value	%																																																																																							
0																																																																																								

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																															
NN	Lactat	IL	Lact-d																																						
Calibration			Condition			Calibration/R-Blank Limit																																			
<table border="1"> <thead> <tr> <th colspan="2">Method</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td></td> </tr> </tbody> </table>			Method		Value	Unit	1-point		<table border="1"> <thead> <tr> <th colspan="2">Curve Type</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>Linear</td> <td></td> </tr> </tbody> </table>			Curve Type		Value	Unit	Linear		<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table>						R-Blank Limit		Value	Unit	2500.0	mAbs												
Method																																									
Value	Unit																																								
1-point																																									
Curve Type																																									
Value	Unit																																								
Linear																																									
R-Blank Limit																																									
Value	Unit																																								
2500.0	mAbs																																								
<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> </tr> </tbody> </table>			Repeats		Value	Unit	2		<table border="1"> <thead> <tr> <th colspan="2">Stability</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td></td> <td>days</td> </tr> </tbody> </table>			Stability		Value	Unit		days	<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>20.0</td> <td>%</td> </tr> </tbody> </table>						Cal Reps Range		Value	Unit	20.0	%												
Repeats																																									
Value	Unit																																								
2																																									
Stability																																									
Value	Unit																																								
	days																																								
Cal Reps Range																																									
Value	Unit																																								
20.0	%																																								
<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			No		Value	Unit			<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>100.0</td> <td>mAbs</td> </tr> </tbody> </table>			Min Cal Reps		Value	Unit	100.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>20</td> <td>%</td> </tr> </tbody> </table>						Factor Change		Value	Unit	20	%												
No																																									
Value	Unit																																								
Min Cal Reps																																									
Value	Unit																																								
100.0	mAbs																																								
Factor Change																																									
Value	Unit																																								
20	%																																								
<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCal **</td> </tr> <tr> <td>2</td> <td></td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>			Calibrator		Calibrator	Conc	1	TruCal **	2		3		4		5		<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <th>Value</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>			M-Point Curve Fit		Value	Unit	0.0	%	<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> <tr> <th>Condition</th> <th>Value</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/> Reagent Blank</td> <td></td> </tr> <tr> <td><input type="checkbox"/> Auto Reagent Blank by Bottle</td> <td></td> </tr> <tr> <td><input type="checkbox"/> Ask for calibration when reagent lot changes</td> <td></td> </tr> </tbody> </table>						Condition		Condition	Value	<input checked="" type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes	
Calibrator																																									
Calibrator	Conc																																								
1	TruCal **																																								
2																																									
3																																									
4																																									
5																																									
M-Point Curve Fit																																									
Value	Unit																																								
0.0	%																																								
Condition																																									
Condition	Value																																								
<input checked="" type="checkbox"/> Reagent Blank																																									
<input type="checkbox"/> Auto Reagent Blank by Bottle																																									
<input type="checkbox"/> Ask for calibration when reagent lot changes																																									

* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: линейная (калибратор TruCal U).

Контроль по TruLab N и TruLab P.