

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **Mg** Test Code: IL_ **Mg-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
☒ Standard ☐ Extended

Reporting Unit
☒ ммоль/л * **Decimal Points** 2
☐ User Define
Conv.Factor 0.000

Methodology
Type
☒ End Point ☐ Rate
Measuring Point 8 -
Photometric
☐ 1 Wavelength ☒ 2 Wavelength
Primary 546 **Secondary** 700
Correction Constant
Slope 1.000 **Intercept** 0.000

Test Name: NN **Mg** Test Code: IL_ **Mg-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample Volume	Dilution Sample Vol.	Dilution Diluent Vol.
1	2.0	0.0	0.0
2	2.0	20.0	80.0
3	4.0	0.0	0.0
4			

Diluent
Diluent Warning Limit 0 Tests

Reagent Volume
R1 **Mg-d** **R2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	<input checked="" type="checkbox"/>	20	No Control
R2			<input type="checkbox"/>		

Sampling Condition

Condition No.	1	2	3	4
First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Ranges			Limits																																								
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.73</td> <td>1.06</td> </tr> <tr> <td>Female</td> <td>0.77</td> <td>1.03</td> </tr> <tr> <td>Other</td> <td>0.73</td> <td>1.06</td> </tr> </tbody> </table>			Normal Range				Lower	Upper	Male	0.73	1.06	Female	0.77	1.03	Other	0.73	1.06	<table border="1"> <thead> <tr> <th>Reaction Slope</th> <th>Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td> <input type="radio"/> Negative <input checked="" type="radio"/> Positive </td> <td> <input checked="" type="radio"/> Above <input type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td colspan="2"> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table></td></tr></tbody> </table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		0	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td colspan="2"> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>	Judge Point		0	
Normal Range																																											
	Lower	Upper																																									
Male	0.73	1.06																																									
Female	0.77	1.03																																									
Other	0.73	1.06																																									
Reaction Slope	Absorbance Limit																																										
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																										
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead></table>	Non-Linear Limit		0	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td colspan="2"> <input type="radio"/> Above <input checked="" type="radio"/> Below </td> </tr> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>	Judge Point		0																									
Non-Linear Limit																																											
0	%																																										
Prozone Limit																																											
<input type="radio"/> Above <input checked="" type="radio"/> Below																																											
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>	Judge Point		0																																			
Limit	Equation																																										
0.0	none																																										
Judge Point																																											
0																																											
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.02</td> <td>2.05</td> </tr> </tbody> </table>			User Range		Lower	Upper	0.02	2.05																																			
User Range																																											
Lower	Upper																																										
0.02	2.05																																										
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.02</td> <td>2.05</td> </tr> </tbody> </table>			ValidRange		Lower	Upper	0.02	2.05																																			
ValidRange																																											
Lower	Upper																																										
0.02	2.05																																										
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>			Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																															
Serum Index Limits																																											
	Limit																																										
Hemolysis	0.0																																										
Icterus	0.0																																										
Lipemia	0.0																																										
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> <tr> <td colspan="2"> <input type="radio"/> On <input checked="" type="radio"/> Off </td> </tr> </thead></table>			Qualitative		<input type="radio"/> On <input checked="" type="radio"/> Off																																						
Qualitative																																											
<input type="radio"/> On <input checked="" type="radio"/> Off																																											

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																																																					
NN	Mg	IL	Mg-d																																																																												
Calibration				Condition				Calibration/R-Blank Limit																																																																							
<table border="1"> <thead> <tr> <th>Method</th> <th>Curve Type</th> </tr> </thead> <tbody> <tr> <td>1-point</td> <td>Linear</td> </tr> </tbody> </table>				Method	Curve Type	1-point	Linear	<table border="1"> <thead> <tr> <th>Repeats</th> <th>Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td>days</td> </tr> </tbody> </table>				Repeats	Stability	2	days	<table border="1"> <thead> <tr> <th colspan="2">Calibrator</th> </tr> <tr> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>TruCalU **</td> </tr> <tr> <td>2</td> <td></td> </tr> <tr> <td>3</td> <td></td> </tr> <tr> <td>4</td> <td></td> </tr> <tr> <td>5</td> <td></td> </tr> </tbody> </table>				Calibrator		Calibrator	Conc	1	TruCalU **	2		3		4		5		<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>50.0</td> <td>mAbs</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table> </td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table>		Cal Reps Range		20.0	%	<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>50.0</td> <td>mAbs</td> </tr> </thead></table>		Min Cal Reps		50.0	mAbs	<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>		Factor Change		20	%	<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table>		M-Point Curve Fit		0.0	%	<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> <tr> <th></th> <th></th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition				<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes
Method	Curve Type																																																																														
1-point	Linear																																																																														
Repeats	Stability																																																																														
2	days																																																																														
Calibrator																																																																															
Calibrator	Conc																																																																														
1	TruCalU **																																																																														
2																																																																															
3																																																																															
4																																																																															
5																																																																															
R-Blank Limit																																																																															
2500.0	mAbs																																																																														
<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>20.0</td> <td>%</td> </tr> </thead></table>		Cal Reps Range		20.0	%																																																																										
Cal Reps Range																																																																															
20.0	%																																																																														
<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>50.0</td> <td>mAbs</td> </tr> </thead></table>		Min Cal Reps		50.0	mAbs																																																																										
Min Cal Reps																																																																															
50.0	mAbs																																																																														
<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>20</td> <td>%</td> </tr> </thead></table>		Factor Change		20	%																																																																										
Factor Change																																																																															
20	%																																																																														
<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead></table>		M-Point Curve Fit		0.0	%																																																																										
M-Point Curve Fit																																																																															
0.0	%																																																																														
Condition																																																																															
<input checked="" type="checkbox"/>	Reagent Blank																																																																														
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																																																														
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																																																														

Магний

Xylidil blue (метод с ксилидиновым синим)



* -Рекомендуемый параметр.

****** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) или по стандарту из набора.

Контроль по TruLab N и TruLab P.