

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **Transfer** Test Code: IL_ **Trf-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Type
Serum

Reaction Cycle
Standard Extended

Reporting Unit
г/л* Decimal Points: 2
User Define
Conv. Factor: 0.000

Methodology
Type: End Point Rate
Measuring Point: 16 - 33
Photometric: 1 Wavelength 2 Wavelength
Primary: 570 Secondary:
Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN **Transfer** Test Code: IL_ **Trf-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

Sample Volume

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	2.0	0.0	0.0
2	2.0	30.0	60.0
3	4.0	0.0	0.0
4	0.0	0.0	0.0

Diluent:
Diluent Warning Limit: 0 Tests

Reagent Volume

R1: Trf-d1 R2: Trf-d2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	250	0.0	✓	20	No Control
R2	50	0.0	✓	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
First Run		✓			
Samp.Vol.Reduction			✓		
Below N-Range					
Above N-Range					
Panic L					
Panic H					
User Range L					
User Range H	✓		✓		
Noise					
Prozone					
HIGH!	✓		✓		
ABS!	✓		✓		

Ranges		Limits																																																																																						
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>2.0</td> <td>3.6</td> </tr> <tr> <td>Female</td> <td>2.0</td> <td>3.6</td> </tr> <tr> <td>Other</td> <td>2.0</td> <td>3.6</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	2.0	3.6	Female	2.0	3.6	Other	2.0	3.6	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead></table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead></table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Limit</th> <th colspan="2">Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td></td> <td>none</td> <td></td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table> </td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead></table>		Non-Linear Limit		100	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead></table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Limit</th> <th colspan="2">Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td></td> <td>none</td> <td></td> </tr> </tbody> </table>		Limit		Equation		0.0		none		<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0	
Normal Range																																																																																								
	Lower	Upper																																																																																						
Male	2.0	3.6																																																																																						
Female	2.0	3.6																																																																																						
Other	2.0	3.6																																																																																						
Reaction Slope		Absorbance Limit																																																																																						
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																					
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead></table>		Non-Linear Limit		100	%	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead></table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																													
Non-Linear Limit																																																																																								
100	%																																																																																							
Prozone Limit																																																																																								
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">Limit</th> <th colspan="2">Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td></td> <td>none</td> <td></td> </tr> </tbody> </table>		Limit		Equation		0.0		none																																																																				
Serum Index Limits																																																																																								
	Limit																																																																																							
Hemolysis	0.0																																																																																							
Icterus	0.0																																																																																							
Lipemia	0.0																																																																																							
Limit		Equation																																																																																						
0.0		none																																																																																						
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.01	8.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.01</td> <td>8.0</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.01	8.0																																																																									
User Range																																																																																								
Lower	Upper																																																																																							
0.01	8.0																																																																																							
ValidRange																																																																																								
Lower	Upper																																																																																							
0.01	8.0																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>		Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead></table>		Judge Point		0																																																																														
Qualitative																																																																																								
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																							
Judge Point																																																																																								
0																																																																																								

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																					
NN	Transfer	IL	Trf-d																																												
Calibration				Condition				Calibration/R-Blank Limit																																							
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>5P Log</td> <td></td> </tr> </tbody> </table>				Method		Curve Type		M-point		5P Log		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>				Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td>6</td> <td></td> </tr> </tbody> </table>				No		6																	
Method		Curve Type																																													
M-point		5P Log																																													
Repeats		Stability																																													
2			days																																												
No																																															
6																																															
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl*</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-Prot-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-Prot-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-Prot-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-Prot-4</td> <td>**</td> </tr> <tr> <td>6</td> <td>TC-Prot-5</td> <td>**</td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-Prot-1	**	3	TC-Prot-2	**	4	TC-Prot-3	**	5	TC-Prot-4	**	6	TC-Prot-5	**	<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		<input type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </tbody> </table>				R-Blank Limit		2500.0	mAbs
Calibrator																																															
	Calibrator	Conc																																													
1	NaCl*	0.0																																													
2	TC-Prot-1	**																																													
3	TC-Prot-2	**																																													
4	TC-Prot-3	**																																													
5	TC-Prot-4	**																																													
6	TC-Prot-5	**																																													
Condition																																															
<input type="checkbox"/>	Reagent Blank																																														
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																														
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																														
R-Blank Limit																																															
2500.0	mAbs																																														
				<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> </thead> <tbody> <tr> <td>15.0</td> <td>%</td> </tr> </tbody> </table>				Cal Reps Range		15.0	%	<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> </thead> <tbody> <tr> <td>150.0</td> <td>mAbs</td> </tr> </tbody> </table>				Min Cal Reps		150.0	mAbs																												
Cal Reps Range																																															
15.0	%																																														
Min Cal Reps																																															
150.0	mAbs																																														
				<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> </thead> <tbody> <tr> <td>35</td> <td>%</td> </tr> </tbody> </table>				Factor Change		35	%	<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>				M-Point Curve Fit		0.0	%																												
Factor Change																																															
35	%																																														
M-Point Curve Fit																																															
0.0	%																																														

Трансферрин

ИттипoTD (Иммунотурбидиметрический)



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору1

Калибровка многоточечная, (6 точек, 0+5 уровней) Spline или 5P Log, калибратор TruCal Protein.

Контроль TruLab Protein уровень 1 и 2.