

## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN **BiT** Test Code: IL\_ **BiT-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

**Sample Type**  
**Serum**

**Reaction Cycle**  
☒ Standard ☐ Extended

**Reporting Unit**  
☒ **МКМОЛЬ/Л** \* **Decimal Points** 1  
☐ User Define  
Conv.Factor: 0.000

**Methodology**  
**Type**  
☒ End Point ☐ Rate  
**Measuring Point**  
16 - ☒ 33  
**Photometric**  
☐ 1 Wavelength ☒ 2 Wavelength  
**Primary** 546 **Secondary** 700  
**Correction Constant**  
**Slope** 1.000 **Intercept** 0.000

Test Name: NN **BiT** Test Code: IL\_ **BiT-d** **Measure** **Sample Reagent** **Ranges Limits** **Calibration**

**Sample Volume**

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	4.0	0.0	0.0
2	2.0	0.0	0.0
3	8.0	0.0	0.0
4	0.0	0.0	0.0

**Diluent** **Diluent Warning Limit** 0 Tests

**Reagent Volume**  
**R1** **BiT-d1** **R2** **BiT-d2**

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	160	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	40	0.0	<input checked="" type="checkbox"/>	20	No Control

**Sampling Condition**

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

# Общий билирубин

DCA (метод с дихлоранилином (ДХА))

Ranges	Limits																																																																											
<table><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>1.7</td><td>21</td></tr><tr><td>Female</td><td>1.7</td><td>21</td></tr><tr><td>Other</td><td>1.7</td><td>21</td></tr></tbody></table> <table><thead><tr><th colspan="2">User Range</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>0</td><td>510</td></tr></tbody></table> <table><thead><tr><th colspan="2">ValidRange</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>0</td><td>510</td></tr></tbody></table>	Normal Range				Lower	Upper	Male	1.7	21	Female	1.7	21	Other	1.7	21	User Range		Lower	Upper	0	510	ValidRange		Lower	Upper	0	510	<table><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table><tr><td>2500.0</td><td>mAbs</td></tr></table></td></tr><tr><td colspan="2"><table><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table></td></tr><tr><td colspan="2"><table><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td><table><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table></td></tr><tr><td><table><thead><tr><th colspan="2">Qualitative</th></tr><tr><td><input type="radio"/> On <input checked="" type="radio"/> Off</td><td></td></tr></thead></table></td><td></td></tr></tbody></table>	Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table><tr><td>2500.0</td><td>mAbs</td></tr></table>	2500.0	mAbs	<table><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td><table><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0			<table><thead><tr><th colspan="2">Qualitative</th></tr><tr><td><input type="radio"/> On <input checked="" type="radio"/> Off</td><td></td></tr></thead></table>	Qualitative		<input type="radio"/> On <input checked="" type="radio"/> Off		
Normal Range																																																																												
	Lower	Upper																																																																										
Male	1.7	21																																																																										
Female	1.7	21																																																																										
Other	1.7	21																																																																										
User Range																																																																												
Lower	Upper																																																																											
0	510																																																																											
ValidRange																																																																												
Lower	Upper																																																																											
0	510																																																																											
Reaction Slope	Absorbance Limit																																																																											
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																																											
<table><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table><tr><td>2500.0</td><td>mAbs</td></tr></table>	2500.0	mAbs																																																																					
Non-Linear Limit																																																																												
0	%																																																																											
2500.0	mAbs																																																																											
<table><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																	
Serum Index Limits																																																																												
	Limit																																																																											
Hemolysis	0.0																																																																											
Icterus	0.0																																																																											
Lipemia	0.0																																																																											
<table><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td><td></td></tr><tr><td><table><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none		<table><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																																																													
Prozone Limit																																																																												
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																																												
<table><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none																																																																								
Limit	Equation																																																																											
0.0	none																																																																											
<table><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																																																																									
Judge Point																																																																												
0																																																																												
<table><thead><tr><th colspan="2">Qualitative</th></tr><tr><td><input type="radio"/> On <input checked="" type="radio"/> Off</td><td></td></tr></thead></table>	Qualitative		<input type="radio"/> On <input checked="" type="radio"/> Off																																																																									
Qualitative																																																																												
<input type="radio"/> On <input checked="" type="radio"/> Off																																																																												

Test Name	Test Code	Measure	Sample Reagent	Ranges Limits	Calibration																																																			
NN	BiT	IL_	BiT-d																																																					
Calibration		Condition		Calibration/R-Blank Limit																																																				
<table><thead><tr><th>Method</th><th>Curve Type</th></tr></thead><tbody><tr><td>1-point</td><td>Linear</td></tr><tr><td>Repeats</td><td>Stability</td></tr><tr><td>2</td><td></td></tr><tr><td>No</td><td></td></tr></tbody></table> <table><thead><tr><th colspan="2">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>TruCalU</td><td>**</td></tr><tr><td>2</td><td></td><td></td></tr><tr><td>3</td><td></td><td></td></tr><tr><td>4</td><td></td><td></td></tr><tr><td>5</td><td></td><td></td></tr></tbody></table>		Method	Curve Type	1-point	Linear	Repeats	Stability	2		No		Calibrator			Calibrator	Conc	1	TruCalU	**	2			3			4			5			<table><tbody><tr><td><input checked="" type="checkbox"/> Reagent Blank</td><td></td></tr><tr><td><input type="checkbox"/> Auto Reagent Blank by Bottle</td><td></td></tr><tr><td><input type="checkbox"/> Ask for calibration when reagent lot changes</td><td></td></tr></tbody></table>		<input checked="" type="checkbox"/> Reagent Blank		<input type="checkbox"/> Auto Reagent Blank by Bottle		<input type="checkbox"/> Ask for calibration when reagent lot changes		<table><tbody><tr><td>R-Blank Limit</td><td>2500.0</td><td>mAbs</td></tr><tr><td>Cal Reps Range</td><td>20.0</td><td>%</td></tr><tr><td>Min Cal Reps</td><td>50.0</td><td>mAbs</td></tr><tr><td>Factor Change</td><td>20</td><td>%</td></tr><tr><td>M-Point Curve Fit</td><td>0.0</td><td>%</td></tr></tbody></table>		R-Blank Limit	2500.0	mAbs	Cal Reps Range	20.0	%	Min Cal Reps	50.0	mAbs	Factor Change	20	%	M-Point Curve Fit	0.0	%
Method	Curve Type																																																							
1-point	Linear																																																							
Repeats	Stability																																																							
2																																																								
No																																																								
Calibrator																																																								
	Calibrator	Conc																																																						
1	TruCalU	**																																																						
2																																																								
3																																																								
4																																																								
5																																																								
<input checked="" type="checkbox"/> Reagent Blank																																																								
<input type="checkbox"/> Auto Reagent Blank by Bottle																																																								
<input type="checkbox"/> Ask for calibration when reagent lot changes																																																								
R-Blank Limit	2500.0	mAbs																																																						
Cal Reps Range	20.0	%																																																						
Min Cal Reps	50.0	mAbs																																																						
Factor Change	20	%																																																						
M-Point Curve Fit	0.0	%																																																						

# Общий билирубин

*DCA (метод с дихлоранилином (ДХА))*

---



\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) . Контроль по TruLab N и TruLab P.