

# Комплемент C4

ИммуноТД (Иммунотурбидиметрический)

## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN C4
 Test Code: IL\_ C4
 Measure
Sample Reagent
Ranges Limits
Calibration

Sample Type: **Serum**

Reaction Cycle:
   
☒ Standard
 ☐ Extended

Reporting Unit:
   
☒ г/л \*
 ☐ User Define
   
 Decimal Points: 3
   
 Conv.Factor: 0.000

Methodology:
   
 Type:
   
☒ End Point
 ☐ Rate
   
 Measuring Point:
   
 16 - ☒ 33
   
 Photometric:
   
☐ 1 Wavelength
 ☒ 2 Wavelength
   
 Primary: 340
   
 Secondary: 700

Correction Constant:
   
 Slope: 1.000
   
 Intercept: 0.000

Test Name: NN C4
 Test Code: IL\_ C4
 Measure
Sample Reagent
Ranges Limits
Calibration

Sample Volume:
 

	Sample	Dilution	
	Volume	Sample Vol.	Diluent Vol.
1	4.5	0.0	0.0
2	4.0	30.0	60.0
3	6.0	0.0	0.0
4	0.0	0.0	0.0

  
 Diluent: 
  
 Diluent Warning Limit: 0 Tests

Reagent Volume:
   
 R1: C4-1
   
 R2: C4-2
 

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	160	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	32.5	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition:
 

	Condition No.	1	2	3	4
<input type="checkbox"/>	First Run	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Samp.Vol.Reduction	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Below N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Above N-Range	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Panic H	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	User Range L	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	User Range H	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Noise	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/>	Prozone	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	HIGH!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/>	ABS!	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

# Комплемент C4

ИммуноТД (Иммунотурбидиметрический)

Ranges		Limits																																																																																								
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.1</td> <td>0.4</td> </tr> <tr> <td>Female</td> <td>0.1</td> <td>0.4</td> </tr> <tr> <td>Other</td> <td>0.1</td> <td>0.4</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	0.1	0.4	Female	0.1	0.4	Other	0.1	0.4	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.005</td> <td>0.9</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.005</td> <td>0.9</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="4"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		100	%			<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Judge Point		0				<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.005</td> <td>0.9</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.005	0.9	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.005</td> <td>0.9</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.005	0.9			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>				Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off
Normal Range																																																																																										
	Lower	Upper																																																																																								
Male	0.1	0.4																																																																																								
Female	0.1	0.4																																																																																								
Other	0.1	0.4																																																																																								
Reaction Slope		Absorbance Limit																																																																																								
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>100</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>		Non-Linear Limit		100	%			<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Judge Point		0																																																																				
Non-Linear Limit																																																																																										
100	%																																																																																									
Prozone Limit																																																																																										
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																									
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Judge Point		0																																																																																		
Limit	Equation																																																																																									
0.0	none																																																																																									
Judge Point																																																																																										
0																																																																																										
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.005</td> <td>0.9</td> </tr> </tbody> </table>		User Range		Lower	Upper	0.005	0.9	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																							
User Range																																																																																										
Lower	Upper																																																																																									
0.005	0.9																																																																																									
Serum Index Limits																																																																																										
	Limit																																																																																									
Hemolysis	0.0																																																																																									
Icterus	0.0																																																																																									
Lipemia	0.0																																																																																									
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>0.005</td> <td>0.9</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	0.005	0.9																																																																																			
ValidRange																																																																																										
Lower	Upper																																																																																									
0.005	0.9																																																																																									
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>				Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																			
Qualitative																																																																																										
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																									

Test Name		Test Code		Measure	Sample Reagent	Ranges Limits	Calibration																																																																		
NN	C4	IL	C4																																																																						
Calibration		Condition			Calibration/R-Blank Limit																																																																				
<table border="1"> <thead> <tr> <th colspan="2">Method</th> <th colspan="2">Curve Type</th> </tr> </thead> <tbody> <tr> <td>M-point</td> <td></td> <td>5P Log</td> <td></td> </tr> </tbody> </table>		Method		Curve Type		M-point		5P Log		<table border="1"> <thead> <tr> <th colspan="2">Repeats</th> <th colspan="2">Stability</th> </tr> </thead> <tbody> <tr> <td>2</td> <td></td> <td></td> <td>days</td> </tr> </tbody> </table>			Repeats		Stability		2			days	<table border="1"> <thead> <tr> <th colspan="2">No</th> </tr> </thead> <tbody> <tr> <td>6</td> <td></td> </tr> </tbody> </table>			No		6																																															
Method		Curve Type																																																																							
M-point		5P Log																																																																							
Repeats		Stability																																																																							
2			days																																																																						
No																																																																									
6																																																																									
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>NaCl*</td> <td>0.0</td> </tr> <tr> <td>2</td> <td>TC-prot-1</td> <td>**</td> </tr> <tr> <td>3</td> <td>TC-prot-2</td> <td>**</td> </tr> <tr> <td>4</td> <td>TC-prot-3</td> <td>**</td> </tr> <tr> <td>5</td> <td>TC-prot-4</td> <td>**</td> </tr> <tr> <td>6</td> <td>TC-prot-5</td> <td>**</td> </tr> </tbody> </table>		Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-prot-1	**	3	TC-prot-2	**	4	TC-prot-3	**	5	TC-prot-4	**	6	TC-prot-5	**	<table border="1"> <tbody> <tr> <td><input type="checkbox"/> Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/> Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/> Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>			<input type="checkbox"/> Reagent Blank	<input type="checkbox"/> Auto Reagent Blank by Bottle	<input type="checkbox"/> Ask for calibration when reagent lot changes	<table border="1"> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>15.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>200.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>35</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> <td colspan="3"> <table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>			<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	R-Blank Limit		2500.0	mAbs			<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>15.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Cal Reps Range		15.0	%			<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>200.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Min Cal Reps		200.0	mAbs			<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>35</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			Factor Change		35	%			<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			M-Point Curve Fit		0.0	%		
Calibrator																																																																									
	Calibrator	Conc																																																																							
1	NaCl*	0.0																																																																							
2	TC-prot-1	**																																																																							
3	TC-prot-2	**																																																																							
4	TC-prot-3	**																																																																							
5	TC-prot-4	**																																																																							
6	TC-prot-5	**																																																																							
<input type="checkbox"/> Reagent Blank																																																																									
<input type="checkbox"/> Auto Reagent Blank by Bottle																																																																									
<input type="checkbox"/> Ask for calibration when reagent lot changes																																																																									
<table border="1"> <thead> <tr> <th colspan="2">R-Blank Limit</th> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	R-Blank Limit		2500.0	mAbs			<table border="1"> <thead> <tr> <th colspan="2">Cal Reps Range</th> </tr> <tr> <td>15.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Cal Reps Range		15.0	%			<table border="1"> <thead> <tr> <th colspan="2">Min Cal Reps</th> </tr> <tr> <td>200.0</td> <td>mAbs</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>	Min Cal Reps		200.0	mAbs																																																							
R-Blank Limit																																																																									
2500.0	mAbs																																																																								
Cal Reps Range																																																																									
15.0	%																																																																								
Min Cal Reps																																																																									
200.0	mAbs																																																																								
<table border="1"> <thead> <tr> <th colspan="2">Factor Change</th> </tr> <tr> <td>35</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			Factor Change		35	%			<table border="1"> <thead> <tr> <th colspan="2">M-Point Curve Fit</th> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td></td> <td></td> </tr> </tbody> </table>			M-Point Curve Fit		0.0	%																																																										
Factor Change																																																																									
35	%																																																																								
M-Point Curve Fit																																																																									
0.0	%																																																																								

# Комплемент С4

ИттипoTD (Иммунотурбидиметрический)



\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,  
(калибратор TruCal Protein).

Контроль TruLab Protein уровень 1 и 2.