

Креатинин ферментативный

РАР, хинон

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN Creatinin Test Code: IL_ CreaP

Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: ☒ МКМОЛЬ/Л * Decimal Points: 1 ☐ User Define Conv.Factor: 0.000

Methodology

Type: ☒ End Point ☐ Rate

Measuring Point: 16 - ☒ 33

Photometric: ☐ 1 Wavelength ☒ 2 Wavelength

Primary: 546 Secondary: 700

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN Creatinin Test Code: IL_ CreaP

Measure Sample Reagent Ranges Limits Calibration

Sample Volume

	Sample	Dilution
	Volume	Sample Vol. Diluent Vol.
1	4.0	0.0 0.0
2	2.0	0.0 0.0
3	8.0	0.0 0.0
4	0.0	0.0 0.0

Diluent: Diluent Warning Limit: 0 Tests

Reagent Volume

R1: CreaP1 R2: CreaP2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	160	0.0	<input checked="" type="checkbox"/>	20	No Control
R2	80	0.0	<input checked="" type="checkbox"/>	20	No Control

Sampling Condition

	Condition No.	1	2	3	4
<input type="checkbox"/> First Run		<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Samp.Vol.Reduction		<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Below N-Range		<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Above N-Range		<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Panic L		<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Panic H		<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> User Range L		<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> User Range H		<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Noise		<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input type="checkbox"/> Prozone		<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> HIGH!		<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>
<input checked="" type="checkbox"/> ABS!		<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>

Креатинин ферментативный

РАР, хинон

Ranges		Limits																																									
<table border="1"><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>59</td><td>104</td></tr><tr><td>Female</td><td>45</td><td>84</td></tr><tr><td>Other</td><td>45</td><td>104</td></tr></tbody></table>		Normal Range				Lower	Upper	Male	59	104	Female	45	84	Other	45	104	<table border="1"><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table></td><td><table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table></td></tr></tbody></table>		Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none	<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0		
Normal Range																																											
	Lower	Upper																																									
Male	59	104																																									
Female	45	84																																									
Other	45	104																																									
Reaction Slope	Absorbance Limit																																										
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																										
<table border="1"><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>0</td><td>%</td></tr></thead><tbody></tbody></table>	Non-Linear Limit		0	%	<table border="1"><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td><table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table></td></tr><tr><td><table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table></td><td></td></tr></thead></table>	Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none	<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																									
Non-Linear Limit																																											
0	%																																										
Prozone Limit																																											
<input type="radio"/> Above <input checked="" type="radio"/> Below																																											
<table border="1"><thead><tr><th>Limit</th><th>Equation</th></tr></thead><tbody><tr><td>0.0</td><td>none</td></tr></tbody></table>	Limit	Equation	0.0	none																																							
Limit	Equation																																										
0.0	none																																										
<table border="1"><thead><tr><th colspan="2">Judge Point</th></tr><tr><td>0</td><td></td></tr></thead><tbody></tbody></table>	Judge Point		0																																								
Judge Point																																											
0																																											

	User Range			------------	-------		Lower	Upper		0	4000				
	ValidRange			------------	-------		Lower	Upper		0	4000				
Qualitative															
☐ On ☒ Off															

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																						
NN	Creatinin	IL	CreaP																													
Calibration		Condition				Calibration/R-Blank Limit																										
<table border="1"><thead><tr><th colspan="2">Method</th></tr><tr><td>1-point</td><td></td></tr></thead><tbody></tbody></table>		Method		1-point		<table border="1"><thead><tr><th colspan="2">Curve Type</th></tr><tr><td>Linear</td><td></td></tr></thead><tbody></tbody></table>				Curve Type		Linear		<table border="1"><thead><tr><th colspan="2">Reagent Blank</th></tr><tr><td><input checked="" type="checkbox"/></td><td></td></tr></thead><tbody><tr><td><input type="checkbox"/></td><td>Auto Reagent Blank by Bottle</td></tr><tr><td><input type="checkbox"/></td><td>Ask for calibration when reagent lot changes</td></tr></tbody></table>						Reagent Blank		<input checked="" type="checkbox"/>		<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes					
Method																																
1-point																																
Curve Type																																
Linear																																
Reagent Blank																																
<input checked="" type="checkbox"/>																																
<input type="checkbox"/>	Auto Reagent Blank by Bottle																															
<input type="checkbox"/>	Ask for calibration when reagent lot changes																															
<table border="1"><thead><tr><th colspan="2">Repeats</th></tr><tr><td>2</td><td></td></tr></thead><tbody></tbody></table>		Repeats		2		<table border="1"><thead><tr><th colspan="2">Stability</th></tr><tr><td></td><td>days</td></tr></thead><tbody></tbody></table>				Stability			days	<table border="1"><thead><tr><th colspan="2">R-Blank Limit</th></tr><tr><td>2500.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>						R-Blank Limit		2500.0	mAbs									
Repeats																																
2																																
Stability																																
	days																															
R-Blank Limit																																
2500.0	mAbs																															
<table border="1"><thead><tr><th colspan="2">No</th></tr><tr><td></td><td></td></tr></thead><tbody></tbody></table>		No								<table border="1"><thead><tr><th colspan="2">Cal Reps Range</th></tr><tr><td>20.0</td><td>%</td></tr></thead><tbody></tbody></table>						Cal Reps Range		20.0	%													
No																																
Cal Reps Range																																
20.0	%																															
						<table border="1"><thead><tr><th colspan="2">Min Cal Reps</th></tr><tr><td>100.0</td><td>mAbs</td></tr></thead><tbody></tbody></table>						Min Cal Reps		100.0	mAbs																	
Min Cal Reps																																
100.0	mAbs																															
						<table border="1"><thead><tr><th colspan="2">Factor Change</th></tr><tr><td>20</td><td>%</td></tr></thead><tbody></tbody></table>						Factor Change		20	%																	
Factor Change																																
20	%																															
						<table border="1"><thead><tr><th colspan="2">M-Point Curve Fit</th></tr><tr><td>0.0</td><td>%</td></tr></thead><tbody></tbody></table>						M-Point Curve Fit		0.0	%																	
M-Point Curve Fit																																
0.0	%																															
<table border="1"><thead><tr><th colspan="3">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>TruCalU</td><td>**</td></tr><tr><td>2</td><td></td><td></td></tr><tr><td>3</td><td></td><td></td></tr><tr><td>4</td><td></td><td></td></tr><tr><td>5</td><td></td><td></td></tr></tbody></table>												Calibrator				Calibrator	Conc	1	TruCalU	**	2			3			4			5		
Calibrator																																
	Calibrator	Conc																														
1	TruCalU	**																														
2																																
3																																
4																																
5																																

Креатинин ферментативный

РАР, хинон



* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Тип калибровки: Линейная (калибратор TruCalU) . Контроль по TruLab N и TruLab P.