

Имуноглобулин G, Ig-G

ИтипоTD (Иммунотурбидиметрический)

Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN IgG Test Code: IL_ IgG-d

Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: г/л * Decimal Points: 2

User Define: Conv.Factor: 0.000

Methodology: Type: End Point Rate

Measuring Point: 16 - 29

Photometric: 1 Wavelength 2 Wavelength

Primary: 570 Secondary:

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN IgG Test Code: IL_ IgG-d

Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sampling Condition:

Sample	Dilution	Diluent	Condition No.	1	2	3	4
1	2.0	0.0	First Run				
2	2.0	30.0	Samp.Vol.Reduction				
3	4.0	0.0	Below N-Range				
4	0.0	0.0	Above N-Range				
			Panic L				
			Panic H				
			User Range L				
			User Range H				
			Noise				
			Prozone				
			HIGH!				
			ABS!				

Diluent: Diluent Warning Limit: 0 Tests

Reagent Volume: R1: IgG-1 R2: IgG-2

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	350	0.0		20	No Control
R2	75	0.0		20	No Control

Имуноглобулин G, Ig-G

ИммуноTD (Иммунотурбидиметрический)

Ranges	Limits																																																					
<table><thead><tr><th colspan="3">Normal Range</th></tr><tr><th></th><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>Male</td><td>7.0</td><td>16</td></tr><tr><td>Female</td><td>7.0</td><td>16</td></tr><tr><td>Other</td><td>7.0</td><td>16</td></tr></tbody></table>	Normal Range				Lower	Upper	Male	7.0	16	Female	7.0	16	Other	7.0	16	<table><thead><tr><th>Reaction Slope</th><th>Absorbance Limit</th></tr></thead><tbody><tr><td><input type="radio"/> Negative <input checked="" type="radio"/> Positive</td><td><input checked="" type="radio"/> Above <input type="radio"/> Below</td></tr><tr><td><table><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table></td><td><table><tr><td>2600.0</td><td>mAbs</td></tr></table></td></tr><tr><td colspan="2"><table><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table></td></tr><tr><td colspan="2"><table><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit</td><td>Equation</td></tr><tr><td>0.0</td><td>none</td></tr><tr><td colspan="2">Judge Point</td></tr><tr><td colspan="2">0</td></tr></thead></table></td></tr></tbody></table>	Reaction Slope	Absorbance Limit	<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below	<table><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table>	Non-Linear Limit		100	%	<table><tr><td>2600.0</td><td>mAbs</td></tr></table>	2600.0	mAbs	<table><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit</td><td>Equation</td></tr><tr><td>0.0</td><td>none</td></tr><tr><td colspan="2">Judge Point</td></tr><tr><td colspan="2">0</td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit	Equation	0.0	none	Judge Point		0	
Normal Range																																																						
	Lower	Upper																																																				
Male	7.0	16																																																				
Female	7.0	16																																																				
Other	7.0	16																																																				
Reaction Slope	Absorbance Limit																																																					
<input type="radio"/> Negative <input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above <input type="radio"/> Below																																																					
<table><thead><tr><th colspan="2">Non-Linear Limit</th></tr><tr><td>100</td><td>%</td></tr></thead></table>	Non-Linear Limit		100	%	<table><tr><td>2600.0</td><td>mAbs</td></tr></table>	2600.0	mAbs																																															
Non-Linear Limit																																																						
100	%																																																					
2600.0	mAbs																																																					
<table><thead><tr><th colspan="2">Serum Index Limits</th></tr><tr><th></th><th>Limit</th></tr></thead><tbody><tr><td>Hemolysis</td><td>0.0</td></tr><tr><td>Icterus</td><td>0.0</td></tr><tr><td>Lipemia</td><td>0.0</td></tr></tbody></table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																											
Serum Index Limits																																																						
	Limit																																																					
Hemolysis	0.0																																																					
Icterus	0.0																																																					
Lipemia	0.0																																																					
<table><thead><tr><th colspan="2">Prozone Limit</th></tr><tr><td><input type="radio"/> Above <input checked="" type="radio"/> Below</td><td></td></tr><tr><td>Limit</td><td>Equation</td></tr><tr><td>0.0</td><td>none</td></tr><tr><td colspan="2">Judge Point</td></tr><tr><td colspan="2">0</td></tr></thead></table>		Prozone Limit		<input type="radio"/> Above <input checked="" type="radio"/> Below		Limit	Equation	0.0	none	Judge Point		0																																										
Prozone Limit																																																						
<input type="radio"/> Above <input checked="" type="radio"/> Below																																																						
Limit	Equation																																																					
0.0	none																																																					
Judge Point																																																						
0																																																						
<table><thead><tr><th colspan="2">User Range</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>1.25</td><td>35</td></tr></tbody></table>	User Range		Lower	Upper	1.25	35																																																
User Range																																																						
Lower	Upper																																																					
1.25	35																																																					
<table><thead><tr><th colspan="2">ValidRange</th></tr><tr><th>Lower</th><th>Upper</th></tr></thead><tbody><tr><td>1.25</td><td>35</td></tr></tbody></table>	ValidRange		Lower	Upper	1.25	35																																																
ValidRange																																																						
Lower	Upper																																																					
1.25	35																																																					
Qualitative																																																						
<input type="radio"/> On <input checked="" type="radio"/> Off																																																						

Test Name	Test Code	Measure	Sample Reagent	Ranges Limits	Calibration																												
NN	IgG	IL	IgG-d																														
Calibration		Condition		Calibration/R-Blank Limit																													
<table><thead><tr><th>Method</th><th>Curve Type</th></tr></thead><tbody><tr><td>M-point</td><td>5P Log</td></tr><tr><td>Repeats</td><td>Stability</td></tr><tr><td>2</td><td></td></tr><tr><td>No</td><td></td></tr><tr><td>6</td><td>days</td></tr></tbody></table>		Method	Curve Type	M-point	5P Log	Repeats	Stability	2		No		6	days	<table><tbody><tr><td><input type="checkbox"/></td><td>Reagent Blank</td></tr><tr><td><input type="checkbox"/></td><td>Auto Reagent Blank by Bottle</td></tr><tr><td><input type="checkbox"/></td><td>Ask for calibration when reagent lot changes</td></tr></tbody></table>		<input type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table><tbody><tr><td>R-Blank Limit</td></tr><tr><td>2500.0 mAbs</td></tr><tr><td>Cal Reps Range</td></tr><tr><td>15.0 %</td></tr><tr><td>Min Cal Reps</td></tr><tr><td>200.0 mAbs</td></tr><tr><td>Factor Change</td></tr><tr><td>35 %</td></tr><tr><td>M-Point Curve Fit</td></tr><tr><td>0.0 %</td></tr></tbody></table>		R-Blank Limit	2500.0 mAbs	Cal Reps Range	15.0 %	Min Cal Reps	200.0 mAbs	Factor Change	35 %	M-Point Curve Fit	0.0 %
Method	Curve Type																																
M-point	5P Log																																
Repeats	Stability																																
2																																	
No																																	
6	days																																
<input type="checkbox"/>	Reagent Blank																																
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																
R-Blank Limit																																	
2500.0 mAbs																																	
Cal Reps Range																																	
15.0 %																																	
Min Cal Reps																																	
200.0 mAbs																																	
Factor Change																																	
35 %																																	
M-Point Curve Fit																																	
0.0 %																																	
<table><thead><tr><th colspan="3">Calibrator</th></tr><tr><th></th><th>Calibrator</th><th>Conc</th></tr></thead><tbody><tr><td>1</td><td>NaCl*</td><td>0.0</td></tr><tr><td>2</td><td>TC-prot-1</td><td>**</td></tr><tr><td>3</td><td>TC-prot-2</td><td>**</td></tr><tr><td>4</td><td>TC-prot-3</td><td>**</td></tr><tr><td>5</td><td>TC-prot-4</td><td>**</td></tr><tr><td>6</td><td>TC-prot-5</td><td>**</td></tr></tbody></table>		Calibrator				Calibrator	Conc	1	NaCl*	0.0	2	TC-prot-1	**	3	TC-prot-2	**	4	TC-prot-3	**	5	TC-prot-4	**	6	TC-prot-5	**								
Calibrator																																	
	Calibrator	Conc																															
1	NaCl*	0.0																															
2	TC-prot-1	**																															
3	TC-prot-2	**																															
4	TC-prot-3	**																															
5	TC-prot-4	**																															
6	TC-prot-5	**																															

* -Рекомендуемый параметр.

** - Вводится из паспорта калибратору.

Калибровка многоточечная (6 точек, 0+5 уровней) Spline или 5P Log,
(калибратор TruCal Protein).

Контроль TruLab Protein уровень 1 и 2.