

## Параметры для ввода в программу анализатора iLab-Taurus

Test Name: NN TPur Test Code: IL\_ TPur-d

Measure Sample Reagent Ranges Limits Calibration

Sample Type: Serum

Reaction Cycle: Standard Extended

Reporting Unit: Мг /л \* Decimal Points: 2

User Define: Conv.Factor: 0.000

Methodology: Type: End Point Rate

Measuring Point: 33

Photometric: 1 Wavelength 2 Wavelength

Primary: 600 Secondary: 750

Correction Constant: Slope: 1.000 Intercept: 0.000

Test Name: NN TPur Test Code: IL\_ TPur-d

Measure Sample Reagent Ranges Limits Calibration

Sample Volume: Sampling Condition

Sample	Dilution	Diluent	Condition No.	1	2	3	4
1	4.0	0.0	0.0				
2	2.0	0.0	0.0				
3	6.0	0.0	0.0				
4							

Diluent: Diluent Warning Limit: 0 Tests

Reagent Volume: R1: TPur-d R2:

	Volume	Diluent Vol.	stirring	Warning Limit(tests)	Stability(days)
R1	200	0.0	✓	20	No Control
R2					

First Run				
Samp.Vol.Reduction				
Below N-Range				
Above N-Range				
Panic L				
Panic H				
User Range L				
User Range H	✓			
Noise				
Prozone				
HIGH!	✓			
ABS!	✓			

Ranges		Limits																																																																																								
<table border="1"> <thead> <tr> <th colspan="3">Normal Range</th> </tr> <tr> <th></th> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>Male</td> <td>0.0</td> <td>140</td> </tr> <tr> <td>Female</td> <td>0.0</td> <td>140</td> </tr> <tr> <td>Other</td> <td>0.0</td> <td>140</td> </tr> </tbody> </table>		Normal Range				Lower	Upper	Male	0.0	140	Female	0.0	140	Other	0.0	140	<table border="1"> <thead> <tr> <th colspan="2">Reaction Slope</th> <th colspan="2">Absorbance Limit</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> Negative</td> <td><input checked="" type="radio"/> Positive</td> <td><input checked="" type="radio"/> Above</td> <td><input type="radio"/> Below</td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>3500</td> </tr> </tbody> </table> </td> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table> </td> </tr> <tr> <td colspan="2"> <table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>3500</td> </tr> </tbody> </table> </td> <td colspan="2"></td> </tr> <tr> <td colspan="4"> <table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Reaction Slope		Absorbance Limit		<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below	<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Non-Linear Limit		0	%	<input type="text"/>	<input type="text"/>	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>	Judge Point		0		<input type="text"/>	<input type="text"/>	<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>3500</td> </tr> </tbody> </table>		User Range		Lower	Upper	1	3500	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0	<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>3500</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	1	3500			<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>				Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off
Normal Range																																																																																										
	Lower	Upper																																																																																								
Male	0.0	140																																																																																								
Female	0.0	140																																																																																								
Other	0.0	140																																																																																								
Reaction Slope		Absorbance Limit																																																																																								
<input type="radio"/> Negative	<input checked="" type="radio"/> Positive	<input checked="" type="radio"/> Above	<input type="radio"/> Below																																																																																							
<table border="1"> <thead> <tr> <th colspan="2">Non-Linear Limit</th> </tr> <tr> <td>0</td> <td>%</td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>		Non-Linear Limit		0	%	<input type="text"/>	<input type="text"/>	<table border="1"> <thead> <tr> <th colspan="2">Prozone Limit</th> </tr> <tr> <td><input type="radio"/> Above</td> <td><input checked="" type="radio"/> Below</td> </tr> </thead> <tbody> <tr> <td> <table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table> </td> <td> <table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table> </td> </tr> </tbody> </table>		Prozone Limit		<input type="radio"/> Above	<input checked="" type="radio"/> Below	<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>	Judge Point		0		<input type="text"/>	<input type="text"/>																																																																	
Non-Linear Limit																																																																																										
0	%																																																																																									
<input type="text"/>	<input type="text"/>																																																																																									
Prozone Limit																																																																																										
<input type="radio"/> Above	<input checked="" type="radio"/> Below																																																																																									
<table border="1"> <thead> <tr> <th>Limit</th> <th>Equation</th> </tr> </thead> <tbody> <tr> <td>0.0</td> <td>none</td> </tr> </tbody> </table>	Limit	Equation	0.0	none	<table border="1"> <thead> <tr> <th colspan="2">Judge Point</th> </tr> <tr> <td>0</td> <td></td> </tr> </thead> <tbody> <tr> <td><input type="text"/></td> <td><input type="text"/></td> </tr> </tbody> </table>	Judge Point		0		<input type="text"/>	<input type="text"/>																																																																															
Limit	Equation																																																																																									
0.0	none																																																																																									
Judge Point																																																																																										
0																																																																																										
<input type="text"/>	<input type="text"/>																																																																																									
<table border="1"> <thead> <tr> <th colspan="2">User Range</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>3500</td> </tr> </tbody> </table>		User Range		Lower	Upper	1	3500	<table border="1"> <thead> <tr> <th colspan="2">Serum Index Limits</th> </tr> <tr> <th></th> <th>Limit</th> </tr> </thead> <tbody> <tr> <td>Hemolysis</td> <td>0.0</td> </tr> <tr> <td>Icterus</td> <td>0.0</td> </tr> <tr> <td>Lipemia</td> <td>0.0</td> </tr> </tbody> </table>		Serum Index Limits			Limit	Hemolysis	0.0	Icterus	0.0	Lipemia	0.0																																																																							
User Range																																																																																										
Lower	Upper																																																																																									
1	3500																																																																																									
Serum Index Limits																																																																																										
	Limit																																																																																									
Hemolysis	0.0																																																																																									
Icterus	0.0																																																																																									
Lipemia	0.0																																																																																									
<table border="1"> <thead> <tr> <th colspan="2">ValidRange</th> </tr> <tr> <th>Lower</th> <th>Upper</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>3500</td> </tr> </tbody> </table>		ValidRange		Lower	Upper	1	3500																																																																																			
ValidRange																																																																																										
Lower	Upper																																																																																									
1	3500																																																																																									
<table border="1"> <thead> <tr> <th colspan="2">Qualitative</th> </tr> </thead> <tbody> <tr> <td><input type="radio"/> On</td> <td><input checked="" type="radio"/> Off</td> </tr> </tbody> </table>				Qualitative		<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																			
Qualitative																																																																																										
<input type="radio"/> On	<input checked="" type="radio"/> Off																																																																																									

Test Name		Test Code		Measure		Sample Reagent		Ranges Limits		Calibration																																											
NN	TPur	IL	TPur-d																																																		
<table border="1"> <thead> <tr> <th colspan="2">Calibration</th> </tr> </thead> <tbody> <tr> <td>Method</td> <td>Curve Type</td> </tr> <tr> <td>1-point</td> <td>Linear</td> </tr> <tr> <td>Repeats</td> <td>Stability</td> </tr> <tr> <td>2</td> <td></td> </tr> <tr> <td>No</td> <td></td> </tr> </tbody> </table>				Calibration		Method	Curve Type	1-point	Linear	Repeats	Stability	2		No		<table border="1"> <thead> <tr> <th colspan="2">Condition</th> </tr> </thead> <tbody> <tr> <td><input checked="" type="checkbox"/></td> <td>Reagent Blank</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Auto Reagent Blank by Bottle</td> </tr> <tr> <td><input type="checkbox"/></td> <td>Ask for calibration when reagent lot changes</td> </tr> </tbody> </table>				Condition		<input checked="" type="checkbox"/>	Reagent Blank	<input type="checkbox"/>	Auto Reagent Blank by Bottle	<input type="checkbox"/>	Ask for calibration when reagent lot changes	<table border="1"> <thead> <tr> <th colspan="2">Calibration/R-Blank Limit</th> </tr> </thead> <tbody> <tr> <td>R-Blank Limit</td> <td></td> </tr> <tr> <td>2500.0</td> <td>mAbs</td> </tr> <tr> <td>Cal Reps Range</td> <td></td> </tr> <tr> <td>20.0</td> <td>%</td> </tr> <tr> <td>Min Cal Reps</td> <td></td> </tr> <tr> <td>100.0</td> <td>mAbs</td> </tr> <tr> <td>Factor Change</td> <td></td> </tr> <tr> <td>20</td> <td>%</td> </tr> <tr> <td>M-Point Curve Fit</td> <td></td> </tr> <tr> <td>0.0</td> <td>%</td> </tr> </tbody> </table>				Calibration/R-Blank Limit		R-Blank Limit		2500.0	mAbs	Cal Reps Range		20.0	%	Min Cal Reps		100.0	mAbs	Factor Change		20	%	M-Point Curve Fit		0.0	%
Calibration																																																					
Method	Curve Type																																																				
1-point	Linear																																																				
Repeats	Stability																																																				
2																																																					
No																																																					
Condition																																																					
<input checked="" type="checkbox"/>	Reagent Blank																																																				
<input type="checkbox"/>	Auto Reagent Blank by Bottle																																																				
<input type="checkbox"/>	Ask for calibration when reagent lot changes																																																				
Calibration/R-Blank Limit																																																					
R-Blank Limit																																																					
2500.0	mAbs																																																				
Cal Reps Range																																																					
20.0	%																																																				
Min Cal Reps																																																					
100.0	mAbs																																																				
Factor Change																																																					
20	%																																																				
M-Point Curve Fit																																																					
0.0	%																																																				
<table border="1"> <thead> <tr> <th colspan="3">Calibrator</th> </tr> <tr> <th></th> <th>Calibrator</th> <th>Conc</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>StdTPur</td> <td>**</td> </tr> <tr> <td>2</td> <td></td> <td></td> </tr> <tr> <td>3</td> <td></td> <td></td> </tr> <tr> <td>4</td> <td></td> <td></td> </tr> <tr> <td>5</td> <td></td> <td></td> </tr> </tbody> </table>				Calibrator				Calibrator	Conc	1	StdTPur	**	2			3			4			5																															
Calibrator																																																					
	Calibrator	Conc																																																			
1	StdTPur	**																																																			
2																																																					
3																																																					
4																																																					
5																																																					

# Белок в моче

Пирогаллоловый



\* -Рекомендуемый параметр.

\*\* - Вводится из паспорта калибратору.

Тип калибровки: Линейная, по стандарту из набора или по Total protein UC Standard FS (Стандарт общего белка в моче).

Контроль по TruLab Urine уровни 1 и 2.