

Параметры для ввода в программу анализатора Beckman-Coulter AU 480

Parameters		Specific Test Parameters																																																																																																																											
General	LIH	ISE	HbA1c	Calculated Test	Range																																																																																																																								
Test Name: Cyst-C < > Type: Serum Operation Yes																																																																																																																													
<table border="0"> <tr> <td>Sample Volume</td> <td>1 μL</td> <td>Dilution</td> <td>0 μL</td> <td>OD Limit</td> <td></td> </tr> <tr> <td>Pre-Dilution Rate</td> <td>1 ∇</td> <td></td> <td></td> <td>Min.OD</td> <td>-2.0 Max.OD 2.5</td> </tr> <tr> <td>Rgt. Volume</td> <td>R1(R1-1) 90 μL</td> <td>Dilution</td> <td>0 μL</td> <td>Reagent OD Limit</td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>First Low</td> <td>-2.0 High 2.5</td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>Last Low</td> <td>-2.0 High 2.5</td> </tr> <tr> <td></td> <td>R2(R2-1) 30 μL</td> <td>Dilution</td> <td>0 μL</td> <td>Dynamic Range Low</td> <td>0.1 High 8</td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>Correlation Factor A</td> <td>1 B 0</td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>Factor for Maker A</td> <td>1 B 0</td> </tr> <tr> <td>Wavelength</td> <td>Pri 520 ∇nm</td> <td>Sec.</td> <td>NONE ∇nm</td> <td>Onboard Stability Period</td> <td>999 Day Hour</td> </tr> <tr> <td>Method</td> <td>FIXED ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Reaction Slope</td> <td>+ ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Measuring Point1 First</td> <td>12</td> <td>Last</td> <td>24</td> <td></td> <td></td> </tr> <tr> <td>Measuring Point2 First</td> <td></td> <td>Last</td> <td></td> <td></td> <td></td> </tr> <tr> <td>Linearity Limit</td> <td> %</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Lag Time Check</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> </table>						Sample Volume	1 μ L	Dilution	0 μ L	OD Limit		Pre-Dilution Rate	1 ∇			Min.OD	-2.0 Max.OD 2.5	Rgt. Volume	R1(R1-1) 90 μ L	Dilution	0 μ L	Reagent OD Limit						First Low	-2.0 High 2.5					Last Low	-2.0 High 2.5		R2(R2-1) 30 μ L	Dilution	0 μ L	Dynamic Range Low	0.1 High 8					Correlation Factor A	1 B 0					Factor for Maker A	1 B 0	Wavelength	Pri 520 ∇ nm	Sec.	NONE ∇ nm	Onboard Stability Period	999 Day Hour	Method	FIXED ∇					Reaction Slope	+ ∇					Measuring Point1 First	12	Last	24			Measuring Point2 First		Last				Linearity Limit	 %					Lag Time Check	 ∇																																		
Sample Volume	1 μ L	Dilution	0 μ L	OD Limit																																																																																																																									
Pre-Dilution Rate	1 ∇			Min.OD	-2.0 Max.OD 2.5																																																																																																																								
Rgt. Volume	R1(R1-1) 90 μ L	Dilution	0 μ L	Reagent OD Limit																																																																																																																									
				First Low	-2.0 High 2.5																																																																																																																								
				Last Low	-2.0 High 2.5																																																																																																																								
	R2(R2-1) 30 μ L	Dilution	0 μ L	Dynamic Range Low	0.1 High 8																																																																																																																								
				Correlation Factor A	1 B 0																																																																																																																								
				Factor for Maker A	1 B 0																																																																																																																								
Wavelength	Pri 520 ∇ nm	Sec.	NONE ∇ nm	Onboard Stability Period	999 Day Hour																																																																																																																								
Method	FIXED ∇																																																																																																																												
Reaction Slope	+ ∇																																																																																																																												
Measuring Point1 First	12	Last	24																																																																																																																										
Measuring Point2 First		Last																																																																																																																											
Linearity Limit	 %																																																																																																																												
Lag Time Check	 ∇																																																																																																																												
<table border="0"> <tr> <td colspan="2">Parameters</td> <td colspan="4">Specific Test Parameters</td> </tr> <tr> <td>General</td> <td>LIH</td> <td>ISE</td> <td>HbA1c</td> <td>Calculated Test</td> <td>Range</td> </tr> </table>						Parameters		Specific Test Parameters				General	LIH	ISE	HbA1c	Calculated Test	Range																																																																																																												
Parameters		Specific Test Parameters																																																																																																																											
General	LIH	ISE	HbA1c	Calculated Test	Range																																																																																																																								
Test Name: Cyst-C < > Type: Serum																																																																																																																													
<table border="0"> <tr> <td>Value/Flag:</td> <td> ∇</td> <td>Level L:</td> <td></td> <td>Level H:</td> <td></td> </tr> <tr> <td colspan="6">Specific Ranges:</td> </tr> <tr> <td></td> <td>Sex</td> <td>Year</td> <td>Month</td> <td>Year</td> <td>Month</td> </tr> <tr> <td><input type="checkbox"/> 1.</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> 2.</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> 3.</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> 4.</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> 5.</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> 6.</td> <td> ∇</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td colspan="4">7. No demographics</td> <td>0.53</td> <td>1.02</td> </tr> <tr> <td colspan="4">8. Not within expected values</td> <td>0.53</td> <td>1.02</td> </tr> <tr> <td colspan="2">Unit</td> <td colspan="4">mg/l</td> </tr> <tr> <td colspan="2">Decimal Places</td> <td colspan="4">2</td> </tr> </table>						Value/Flag:	 ∇	Level L:		Level H:		Specific Ranges:							Sex	Year	Month	Year	Month	<input type="checkbox"/> 1.	 ∇					<input type="checkbox"/> 2.	 ∇					<input type="checkbox"/> 3.	 ∇					<input type="checkbox"/> 4.	 ∇					<input type="checkbox"/> 5.	 ∇					<input type="checkbox"/> 6.	 ∇					7. No demographics				0.53	1.02	8. Not within expected values				0.53	1.02	Unit		mg/l				Decimal Places		2																																													
Value/Flag:	 ∇	Level L:		Level H:																																																																																																																									
Specific Ranges:																																																																																																																													
	Sex	Year	Month	Year	Month																																																																																																																								
<input type="checkbox"/> 1.	 ∇																																																																																																																												
<input type="checkbox"/> 2.	 ∇																																																																																																																												
<input type="checkbox"/> 3.	 ∇																																																																																																																												
<input type="checkbox"/> 4.	 ∇																																																																																																																												
<input type="checkbox"/> 5.	 ∇																																																																																																																												
<input type="checkbox"/> 6.	 ∇																																																																																																																												
7. No demographics				0.53	1.02																																																																																																																								
8. Not within expected values				0.53	1.02																																																																																																																								
Unit		mg/l																																																																																																																											
Decimal Places		2																																																																																																																											
<table border="0"> <tr> <td colspan="2">Parameters</td> <td colspan="4">Calibration Parameters</td> </tr> <tr> <td>Calibrators</td> <td></td> <td>Calibration Specific</td> <td colspan="3">STAT Table Calibration</td> </tr> <tr> <td>General</td> <td>ISE</td> <td colspan="4"></td> </tr> </table>						Parameters		Calibration Parameters				Calibrators		Calibration Specific	STAT Table Calibration			General	ISE																																																																																																										
Parameters		Calibration Parameters																																																																																																																											
Calibrators		Calibration Specific	STAT Table Calibration																																																																																																																										
General	ISE																																																																																																																												
Test Name: Cyst-C < > Type: Serum ∇ o Use Serum Cal.																																																																																																																													
<table border="0"> <tr> <td colspan="2">Calibration Type: 6AB ∇</td> <td colspan="2">Formula: Spline ∇</td> <td colspan="2">Counts: 2 ∇</td> </tr> <tr> <td colspan="6"><Calibrator Parameters></td> </tr> <tr> <td>Calibrator</td> <td>OD</td> <td>Conc</td> <td>Low</td> <td>High</td> <td></td> </tr> <tr> <td>Point 1:</td> <td>Saline</td> <td>0</td> <td>-0.1</td> <td>2.5</td> <td></td> </tr> <tr> <td>Point 2:</td> <td>TruCal Cyst-C1</td> <td>*</td> <td>-0.1</td> <td>2.5</td> <td></td> </tr> <tr> <td>Point 3:</td> <td>TruCal Cyst-C2</td> <td>*</td> <td>-0.1</td> <td>2.5</td> <td></td> </tr> <tr> <td>Point 4:</td> <td>TruCal Cyst-C3</td> <td>*</td> <td>-0.1</td> <td>2.5</td> <td></td> </tr> <tr> <td>Point 5:</td> <td>TruCal Cyst-C4</td> <td>*</td> <td>-0.1</td> <td>2.5</td> <td></td> </tr> <tr> <td>Point 6:</td> <td>TruCal Cyst-C5</td> <td>*</td> <td>-0.1</td> <td>2.5</td> <td></td> </tr> <tr> <td>Point 7:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 8:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 9:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Point 10:</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td colspan="6"> <table border="0"> <tr> <td><Point Cal. For</td> <td>No. of Correction Points</td> <td> ∇</td> <td>Use Master Curve</td> <td> ∇</td> <td>o Lot Calibration</td> </tr> <tr> <td colspan="6"><Master Curve></td> </tr> <tr> <td>Calibrator</td> <td>OD</td> <td>Conc</td> <td>Low</td> <td>High</td> <td>Stability</td> </tr> <tr> <td>Point 1:</td> <td></td> <td></td> <td></td> <td></td> <td>Reagent Blank Day Hour</td> </tr> <tr> <td>Point 2:</td> <td></td> <td></td> <td></td> <td></td> <td>Calibration Day Hour</td> </tr> </table> </td> </tr> <tr> <td colspan="6"> MB Type Factor: 1-Point Calibration Point ∇ o with Conc-0 </td> </tr> </table>						Calibration Type: 6AB ∇		Formula: Spline ∇		Counts: 2 ∇		<Calibrator Parameters>						Calibrator	OD	Conc	Low	High		Point 1:	Saline	0	-0.1	2.5		Point 2:	TruCal Cyst-C1	*	-0.1	2.5		Point 3:	TruCal Cyst-C2	*	-0.1	2.5		Point 4:	TruCal Cyst-C3	*	-0.1	2.5		Point 5:	TruCal Cyst-C4	*	-0.1	2.5		Point 6:	TruCal Cyst-C5	*	-0.1	2.5		Point 7:						Point 8:						Point 9:						Point 10:						<table border="0"> <tr> <td><Point Cal. For</td> <td>No. of Correction Points</td> <td> ∇</td> <td>Use Master Curve</td> <td> ∇</td> <td>o Lot Calibration</td> </tr> <tr> <td colspan="6"><Master Curve></td> </tr> <tr> <td>Calibrator</td> <td>OD</td> <td>Conc</td> <td>Low</td> <td>High</td> <td>Stability</td> </tr> <tr> <td>Point 1:</td> <td></td> <td></td> <td></td> <td></td> <td>Reagent Blank Day Hour</td> </tr> <tr> <td>Point 2:</td> <td></td> <td></td> <td></td> <td></td> <td>Calibration Day Hour</td> </tr> </table>						<Point Cal. For	No. of Correction Points	 ∇	Use Master Curve	 ∇	o Lot Calibration	<Master Curve>						Calibrator	OD	Conc	Low	High	Stability	Point 1:					Reagent Blank Day Hour	Point 2:					Calibration Day Hour	MB Type Factor: 1-Point Calibration Point ∇ o with Conc-0					
Calibration Type: 6AB ∇		Formula: Spline ∇		Counts: 2 ∇																																																																																																																									
<Calibrator Parameters>																																																																																																																													
Calibrator	OD	Conc	Low	High																																																																																																																									
Point 1:	Saline	0	-0.1	2.5																																																																																																																									
Point 2:	TruCal Cyst-C1	*	-0.1	2.5																																																																																																																									
Point 3:	TruCal Cyst-C2	*	-0.1	2.5																																																																																																																									
Point 4:	TruCal Cyst-C3	*	-0.1	2.5																																																																																																																									
Point 5:	TruCal Cyst-C4	*	-0.1	2.5																																																																																																																									
Point 6:	TruCal Cyst-C5	*	-0.1	2.5																																																																																																																									
Point 7:																																																																																																																													
Point 8:																																																																																																																													
Point 9:																																																																																																																													
Point 10:																																																																																																																													
<table border="0"> <tr> <td><Point Cal. For</td> <td>No. of Correction Points</td> <td> ∇</td> <td>Use Master Curve</td> <td> ∇</td> <td>o Lot Calibration</td> </tr> <tr> <td colspan="6"><Master Curve></td> </tr> <tr> <td>Calibrator</td> <td>OD</td> <td>Conc</td> <td>Low</td> <td>High</td> <td>Stability</td> </tr> <tr> <td>Point 1:</td> <td></td> <td></td> <td></td> <td></td> <td>Reagent Blank Day Hour</td> </tr> <tr> <td>Point 2:</td> <td></td> <td></td> <td></td> <td></td> <td>Calibration Day Hour</td> </tr> </table>						<Point Cal. For	No. of Correction Points	 ∇	Use Master Curve	 ∇	o Lot Calibration	<Master Curve>						Calibrator	OD	Conc	Low	High	Stability	Point 1:					Reagent Blank Day Hour	Point 2:					Calibration Day Hour																																																																																										
<Point Cal. For	No. of Correction Points	 ∇	Use Master Curve	 ∇	o Lot Calibration																																																																																																																								
<Master Curve>																																																																																																																													
Calibrator	OD	Conc	Low	High	Stability																																																																																																																								
Point 1:					Reagent Blank Day Hour																																																																																																																								
Point 2:					Calibration Day Hour																																																																																																																								
MB Type Factor: 1-Point Calibration Point ∇ o with Conc-0																																																																																																																													

Диапазон нормальных значений указан в соответствии с рекомендациями производителя.

При использовании единиц измерения, отличающихся от приведенных, убедитесь, что значения стандартов, контрольных материалов, диапазонов нормальных значений и линейности метода введены в этих же единицах.

* -вводится из паспорта к калибратору (TruCal Cystatin C).

Контроль по TruLab Cystatin уровень 1 и уровень 2.